








Ostial

FLASH™

Aorto-Ostial Angioplasty System

TRUE

360°

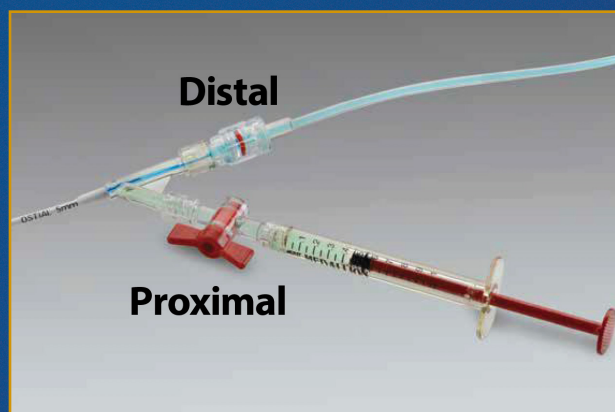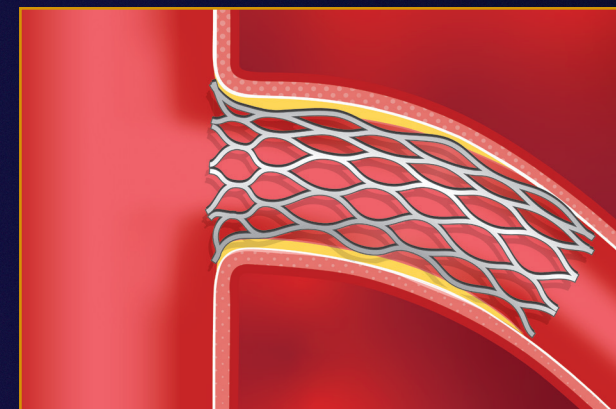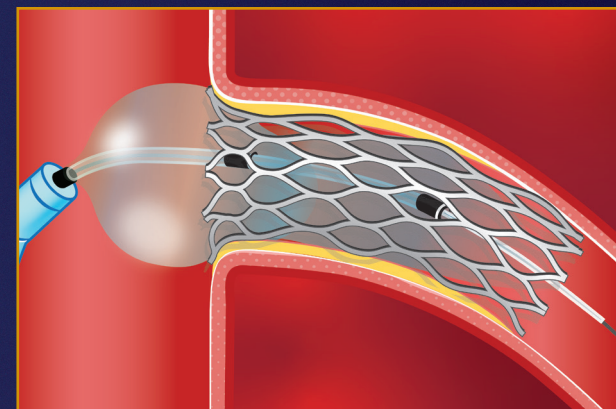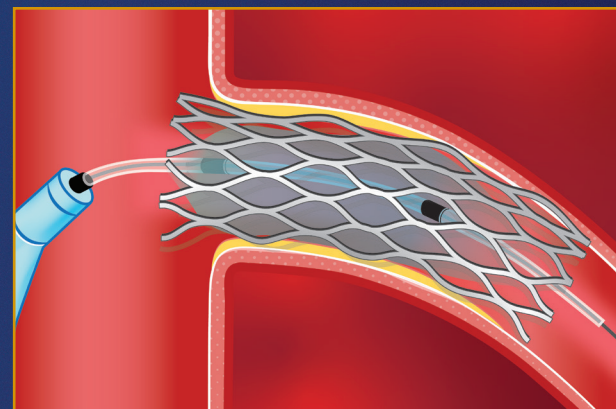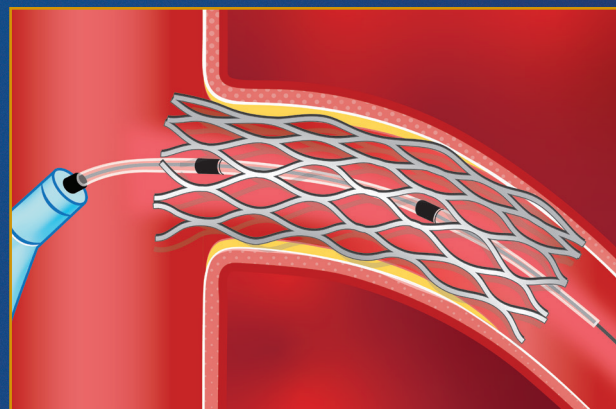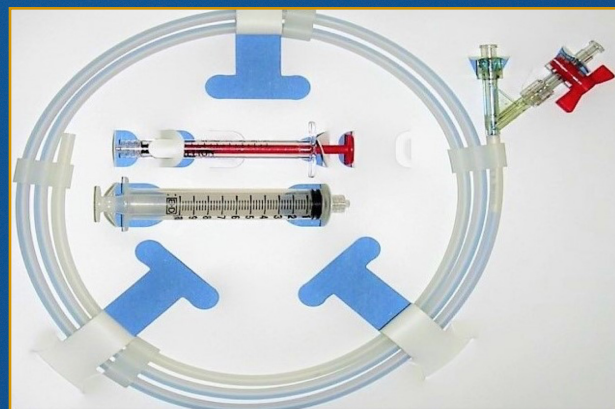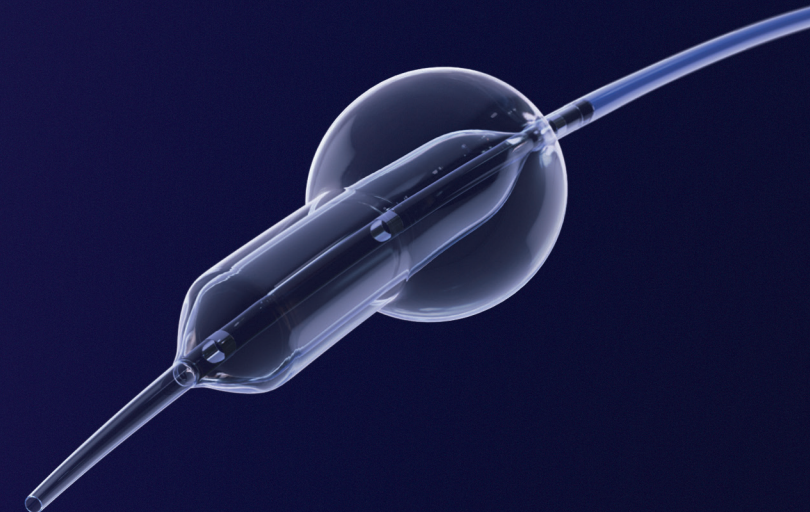

PREP THE DEVICE

Proximal Balloon: Prep and purge using a 10 cc or 20 cc syringe, then attach the 1 cc syringe (red), filled with 50/50 contrast/saline mix.

Distal Balloon: Use an inflator to prep, similar to a regular angioplasty balloon.

ALIGN MARKERS

Mid-marker: At the ostium (placed 2 mm distal to the proximal edge of the stent).

Distal marker: Ensure at least 2–3 mm within the stent.

Proximal marker: In the aorta, outside the guide catheter.

INFLATE DISTAL BALLOON

Inflate the distal balloon with an inflator to the desired pressure.

INFLATE PROXIMAL BALLOON

Before inflation of the proximal balloon, make sure the distal marker has not moved and is inside the stent. Using 50/50 contrast, inflate the proximal balloon using the red 1 cc inflation syringe.

Inflation volume:

FLASH™ = 1 cc

FLASH™ Mini = 0.4 cc

TRUE 360° RESULT

Deflate the proximal balloon with a 10 cc or 20 cc syringe and deflate the distal balloon with the inflator. Withdraw the FLASH System.



Play FLASH
Training Video